OWNER'S MANUAL

Model AC017

Baby Breathing Monitor

TABLE OF CONTENTS

 WARNINGS	2
SETUP REQUIREMENTS.....	3
Setting up the Nursery Unit.....	4
Positioning the movement Sensor Pad.....	4
Turning ON your monitor.....	5
Getting to know your monitor.....	6
Troubleshooting.....	7
Limited warranty.....	8
Technical specifications	8
FRENCH	9
SPANISH	17

OWNER'S MANUAL

KEEP YOUR OWNER'S MANUAL FOR FUTURE REFERENCE.

PLEASE READ THE WARNINGS AND INSTRUCTIONS CAREFULLY BEFORE ASSEMBLING & USING THE PRODUCT.

FAILURE TO FOLLOW THE WARNINGS AND THE INSTRUCTIONS COULD RESULT IN SERIOUS INJURY OR DEATH OR DAMAGES TO THE PRODUCT.

THE SENSOR PAD FEATURE IS NOT FOR USE WITH ADULTS, CHILDREN OR INFANTS OLDER THAN 24 MONTHS.

THE SENSOR PAD IS NOT INTENDED TO BE USED TO DETERMINE IF A CHILD IS OUT OF THE CRIB.

WARNING

Adult assembly required.

Keep small parts away from children when assembling.

We, at Angelcare®, stand behind all of our products. If you are not completely satisfied or have any questions,

United States and Canada: Please contact us at:

angelcare@cdceinc.ca

Other countries: Please contact your local distributor.

FCC AND IC REGULATIONS

RF radiation exposure statement

The Nursery Unit complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. **Caution:** To maintain the compliance with the FCC's and Industry Canada's RF exposure guideline, install the Nursery Unit at least 7.9" (20 cm) from ALL persons. The transmitter of this device must not be co-located or operating in conjunction with any other antenna or transmitter.

Industry Canada

This product meets the applicable Industry Canada technical specifications. CAN ICES-3 (B)/NMB-3 (B)

WARNINGS

WARNING -

- **KEEP** the Nursery unit **out** of the reach of children more than 3 feet or 1 meter away.
- **This product is NOT a Toy. NEVER allow children to play with it.**

1. THIS PRODUCT CANNOT REPLACE RESPONSIBLE ADULT SUPERVISION. This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional. This monitor SHOULD NOT be used as a medical device, nor as a device to prevent Sudden Infant Death Syndrome (SIDS).

2. ALWAYS be sure to immediately check on baby whenever an alarm sounds.

3. DO NOT use monitor near water (such as bath tub, sink, etc.).

9.4 Cleaning - Disconnect all units before cleaning. DO NOT immerse in water any part of the monitor. Clean only with dry cloth.

9.5 Ventilation - DO NOT block any ventilation openings. Allow for proper ventilation when units are in use. DO NOT cover the Nursery Unit with any object such as a blanket. DO NOT place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

9.6 Heat sources - DO NOT install near any heat sources such as radiators, heaters, registers, stoves, TVs or other apparatus (including amplifiers) that produce heat. Heat can damage the case or electrical parts.

9.7 Accessories - Only use attachments/accessories specified by the manufacturer. DO NOT place this product on an unstable cart, stand, tripod, bracket or table. The product may fall, causing serious injury to a child or adult and serious damage to the product. Use ONLY with a cart, stand, tripod, bracket or table recommended by the manufacturer or sold with the product. Any mounting of the product should follow the manufacturer's instructions.

9.8 Servicing - DO NOT attempt to service this product yourself as opening or removing covers (except battery door) may expose you to dangerous voltage or other hazards.

9.9 Replacement parts - When replacement parts are required, be sure the replacement parts specified by the manufacturer have the same characteristic as the original part. Unauthorized substitution may result in fire, electric shock or other hazards.

9.10 Usage - This baby monitor is intended as an aid. It is NOT a substitute for responsible and proper adult supervision and should not be used as such. Use this appliance ONLY for its intended use as described in this manual.

9.11 Object and Liquid Entry - NEVER push objects of any kind into this product through casing slots as they may touch dangerous voltage points or short out parts that could result in a risk of fire or electric shock. NEVER spill liquid of any kind on the product.

9.12 Vibration Source - When the monitor is in use, DO NOT use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor. You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned OFF after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with the Angelcare® logo side facing up. The sensitivity level may also need to be adjusted. Refer to **Step 4.3**. Always be sure to immediately check on baby whenever an alarm sounds.

9.13 Sensor Pad - Your Sensor Pad can be used in any crib that is stationary. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4" (6 mm) thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface. The Sensor Pad is made of plastic and does not permit

4. Keep monitor AWAY from heat sources (such as stoves, radiators, etc.).

5. ALWAYS place the Nursery Unit on a flat surface on an upright position and allow for proper ventilation. DO NOT place on sofas, cushions, beds, etc. which may block ventilation.

6. This product contains small parts. Adult assembly required. Exercise care when unpacking and assembling the product and keep small parts away from children.

7. When using the monitor, the room temperature should be between 50°F and 104°F (10°C and 40°C).

BATTERY WARNING

8. The Wireless Sensor Pad battery is a replaceable Button Cell Lithium battery. Only use the Angelcare® recommended replacement battery. Danger of explosion if the wrong battery is used or if replaced incorrectly. Refer to your user manual for correct battery installation. DO NOT short circuit supply terminals. Remove the batteries from your Nursery Unit and Sensor Pad if product is to be stored for an extended period of time. Dispose of batteries according to your local ordinances. The Button cell battery represents a small part and a choking hazard. ALWAYS keepall batteries away from children.

9. IMPORTANT SAFETY INSTRUCTIONS

9.1 Heed all warnings - All warnings on the product and in the operating instructions should be adhered to.

9.2 Follow all instructions - All operating and use instructions should be followed.

9.3 Water and moisture - DO NOT use this apparatus near water. For example, near a bath tub, wash bowl, kitchen sink, laundry tub, in a wet environment or near a swimming pool and the like. Care should be taken so that objects DO NOT fall onto units and that liquids are not spilled into either unit or Sensor Pad through its openings.

the air to circulate. Therefore, mildew build up could develop underneath the mattress in the Sensor Pad area. We suggest rotating the mattress regularly to prevent any mildew.

10. IMPORTANT SAFETY INSTRUCTIONS RELATED TO USE OF BATTERIES

10.1 Use ONLY the type and size of battery specified in the owner's manual. There is a risk of explosion if battery is replaced with an incorrect type.

10.2 DO NOT use rechargeable batteries in the Nursery Unit. Use alkaline batteries ONLY.

10.3 DO NOT dispose batteries in fire. They may explode. Check with local regulations for possible special disposal instructions.

10.4 Exercise care in handling batteries in order not to short out the batteries with conducting materials such as rings, bracelets and keys. Overcharging, short circuiting, reverse charging, mutilation, or incineration of batteries must be avoided to prevent one or more of the following occurrences: release of toxic materials, release of hydrogen and/or oxygen, gas and rise in surface temperature.

10.5 DO NOT attempt to rejuvenate the battery provided with or identified for use with this product by heating them. Sudden release of battery electrolyte may occur, causing burns or irritation to eyes or skin.

10.6 If and when necessary, the battery must be replaced with identical new ones or equivalent (volts and size).

10.7 Reverse polarity installation of the batteries in the end product must be avoided. Reverse insertion of batteries can cause charging, and that may result in leakage or explosion.

10.8 Remove the battery from this product if you store the product over 30 days because the battery could leak and damage the product.

10.9 Discard 'dead' batteries as soon as possible since 'dead' batteries are more likely to leak in a product. If a battery has leaked or vented, it should be replaced immediately using protective gloves.

10.10 DO NOT store this product, or the battery provided with or identified for use with this product, in high-temperature areas.

FC WARNING

(North America Only)

11. Modifications not authorized by the manufacturer may void users authority to operate this device.

11.1 NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

11.2 However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

11.3 This device complies with the part 15 of the FCC Rules and the Industry Canada license - exempt RSS standards(s). Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

IC Warning :


12. This device complies with industry Canada RSS standard(s). Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) This device must accept any interference, including interferences that may cause undesired operation of the device.


12.1 Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

SETUP REQUIREMENTS

INCLUDED: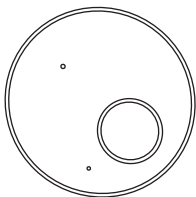

Nursery Unit

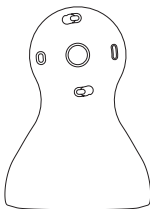Tabletop stand
(for Nursery Unit)

Wireless movement Sensor Pad
* Always place Angelcare® logo side up *

CR3032 battery
(for Sensor Pad)

Model AC017
Baby Breathing Monitor

WHAT YOU WILL NEED (NOT INCLUDED):

Miniature Philips Screwdriver


2 AAA Alkaline
Batteries

STEP 1 - SETTING UP THE NURSERY UNIT


NURSERY UNIT:

Power ON/OFF Light Indicator
Low Battery Light Indicator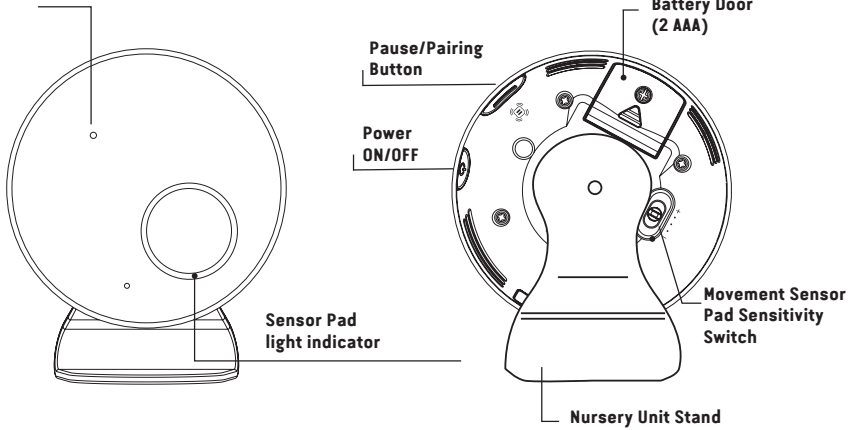

1.1 - INSERT THE BATTERIES


When using the Nursery Unit for the first time, you will need to insert 2 AAA alkaline batteries (not included).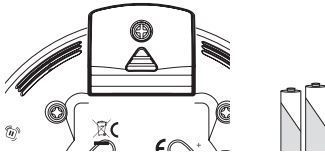

- 1) Remove the battery door with screwdriver and insert 2 AAA batteries.

1.2 - SELECT A LOCATION FOR THE NURSERY UNIT


Nursery Unit can be installed using the tabletop stand.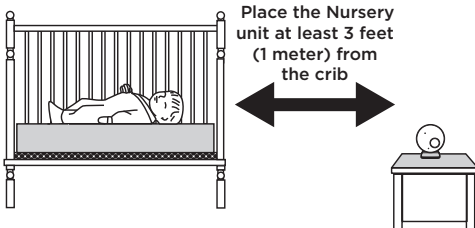

STEP 2 - POSITIONING THE MOVEMENT SENSOR PAD

2.1 - POWERING THE MOVEMENT SENSOR PAD

When using the Sensor Pad for the first time, you will need to insert the provided CR3032 battery.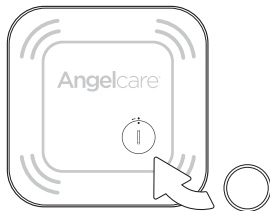

- 1) Remove the battery cover using a coin. Turn in the direction of the arrow.
- 2) Place the included battery (CR3032) in the compartment. **Note:** The + symbol should be facing up.
- 3) Replace the battery cover (the two dots will line up). The Sensor Pad is now **ON**.

2.2 - INSTALL THE SENSOR PAD ON THE MATTRESS SUPPORT

IMPORTANT


If your crib does not have a solid, stable surface for the Sensor Pad, you will need to place a support under it, to ensure correct operation of the system.

This hardboard support should be at least 1/4" (6 mm) thick and should have minimum dimensions of 13" x 13" (33 x 33 cm). This support board must be larger than the Sensor Pad surface area.

The Sensor Pad functions with any size and type of mattress except memory-foam mattresses, hollow frame mattresses and water beds. The hardboard maintains a constant pressure on the Sensor Pad, enabling it to detect the baby's movements even when the baby is at the edges of the crib.

Hardboard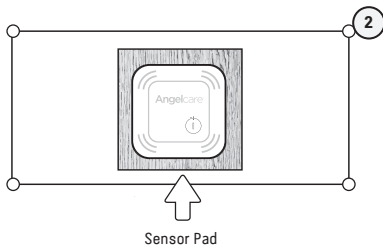


Sensor Pad

- 1) Remove the mattress from the crib.
- 2) Place the Sensor Pad - **Angelcare® logo side up** - in the center of the mattress. It should be centered on the hardboard if your crib requires a hardboard support.
- 3) Replace the mattress.

! WARNING -

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Discontinue used of the Sensor Pad when child is older than 24 months. Remove Sensor Pad from crib and store for future use.
- **BEFORE FIRST TIME USE:** To test that the unit is ready for use, refer to instruction manual on how to test that the installation has been done correctly.

STEP 3 - TURNING ON YOUR MONITOR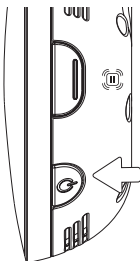

Press and hold the Nursery Unit **ON/OFF** button for 2 seconds until the Nursery Unit Indicator flashes slowly in green.


- The Nursery Unit will emit a sound and flash blue to confirm pairing with the Sensor Pad

NOTE:

If the green light does not appear but the red light is flashing; or if there are false alarms, adjust the sensitivity level (**Step 4.3**).

STEP 4 - GETTING TO KNOW YOUR MONITOR

4.1 TESTING THE SENSOR PAD: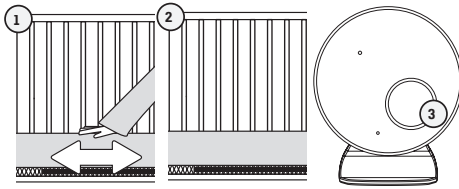

- 1) Move your hand lightly on the mattress.
- 2) Remove your hand from the mattress. After 15 seconds you will hear a pre-alarm sound and the light will flash red.
- 3) 5 seconds later the alarm will emit a loud continuous beeps and the light will flash red continuously.


MOVEMENT ALARM

- If NO movement is detected after 15 seconds you will hear a single pre-alarm sound from the Nursery Unit.
- If after a further 5 seconds there is still NO movement detected, Nursery Unit speakers will emit loud continuous beeps.

IF THE ALARM DOES NOT SOUND


- Avoid contact with crib while Sensor Pad is in operation.
- Place crib near a supporting wall.
- Adjust Sensor Pad sensitivity level.
- Turn **OFF** crib mobile, fans, loud music or anything else that may cause vibration.

TO STOP THE ALARM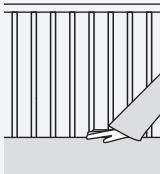

Place your hand back on the mattress.

OR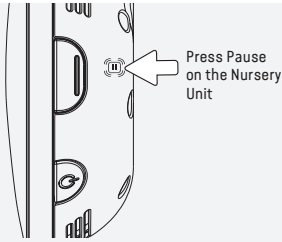


IMPORTANT

- Test your monitor before first use.
- Test your monitor regularly to ensure it is functioning correctly.
- Test your monitor every time you move the Sensor Pad to a different place.

4.2 PAUSING AND RESTARTING THE SENSOR PAD:

Allows you to pause the Sensor Pad if you remove baby from the crib, e.g. for feeding or comforting.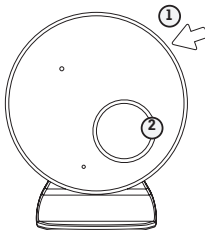

- 1) From the Nursery Unit, press the **Pause** button on the top right of the unit.
- 2) The light will flash white when the Sensor Pad is paused. The Nursery Unit will emit a single beep every 20 seconds to indicate movement function is paused.
- 3) To unpause the Sensor Pad press the **Pause** button again.

IMPORTANT


- If you do not pause the Sensor Pad, the alarm will sound if there is no movement after 20 seconds.
- Ensure to unpause the Sensor Pad before putting baby in the crib.

4.3 - TO ADJUST THE SENSOR PAD SENSITIVITY LEVEL:

To ensure adequate monitoring of your child's movements, you can adjust the sensitivity of the Sensor Pad. Set a high sensitivity for a newborn and a lower sensitivity as the child grows and gains weight. A thicker mattress will also need a higher sensitivity.

Slide the Nursery Unit sensitivity level button to select between low, medium or high level.

- For optimal results begin with the sensitivity level on the button, adjusting the sensitivity by increasing as required.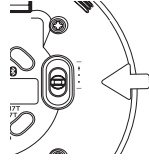

IMPORTANT

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Once you are finished using the Sensor Pad, please remove it from the baby's and store it away.


4.4 - BATTERIES

NURSERY UNIT LOW BATTERY INDICATOR:


The low battery yellow light indicator will flash and an alarm will sound every 20 seconds when the batteries on the Nursery Unit need to be replaced.

SENSOR PAD LOW BATTERY INDICATOR:

The Sensor Pad yellow light Indicator will flash and an alarm will sound every 20 seconds when the battery on the Sensor Pad needs to be replaced.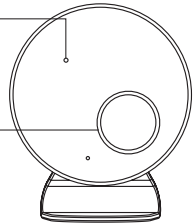

TROUBLESHOOTING

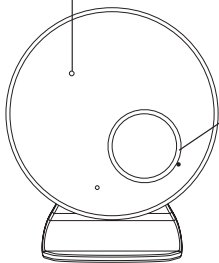Power ON/OFF Light Indicator Low Battery Light Indicator

Flashing green	Monitor is activated
Flashing yellow	Nursery Unit batteries are low

Sensor Pad Light Indicator

Flashing red	No movement is detected or Sensor Pad lost connectivity
Flashing white	Monitor is paused
Flashing yellow	Sensor Pad battery is low

FALSE ALARMS

- Turn **OFF** the Nursery Unit.
- Make sure there is no bedding between the mattress and the Sensor Pad.
- If the crib has a spring or slat base, place a hardboard between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4" (6 mm) thick.
- Make sure Sensor Pad is positioned as described under the setup requirements. Sensitivity may need to be adjusted (refer to **Step 4.3**).
- Pair Sensor Pad and Nursery Unit by pressing and holding the Nursery Unit **Pairing** button for 2 seconds until the light blue flashes and you hear a sound.

ALARM WILL NOT SOUND

- Avoid contact with the crib when the monitor is **ON**.
- Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib.
- Turn **OFF** crib mobile, fans, loud music or anything else that may cause vibration.
- You may need to decrease the sensitivity (refer to **Step 4.3**).

SENSOR PAD LIGHT INDICATOR FLASHING RED

- Make sure there is no bedding between the mattress and the Sensor Pad.
- The Sensor Pad must rest on a completely flat and firm surface. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4" (6 mm) thick.
- Make sure Sensor Pad is positioned as described under the setup requirements. Sensitivity may need to be adjusted (refer to **Step 4.3**).

LOSS OF CONNECTIVITY (NURSERY UNIT BLUE LIGHT FLASHES AND EMITS A SOUND EVERY 20 SECONDS)

- Bring the units closer together.
- Press and hold for 2 seconds the Nursery Unit **Pairing** button, the Nursery Unit will emit a sound and will flash blue to confirm connectivity.

NO POWER

- Check or replace batteries.
- Make sure monitor is **ON**.

LIMITED WARRANTY

Angelcare® warrants to the original owner of this product that Angelcare® monitor is free from any material and workmanship defects for the term of the warranty period (see warranty table below to determine the warranty for your country). If this monitoring system fails to function properly when used as directed under normal conditions and during the warranty period, Angelcare® will either repair or replace the product, at our discretion, free of charge. Please contact your local distributor to validate warranty and warranty period.

The product must be accompanied by a proof of purchase, either a bill of sale or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product which has been damaged as a result of improper maintenance, an accident, improper voltage supply or any other form of misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranty is given.

This warranty does not cover products and accessories which are not Angelcare-branded, or any problems that result from usage that is not in accordance with product instructions, failure to follow the product instructions, or problems caused by using accessories, parts or components not supplied by Angelcare.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM THE USE OF THIS PRODUCT.

IMPORTANT! This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call your distributor before returning this product.

WARRANTY TERM PER COUNTRY

North America	1 year
---------------	--------

TECHNICAL SPECIFICATIONS

Range:

Open field (Open area without obstructions): up to 8 meters.

Power Supply:

2 AAA alkaline batteries for Nursery Unit (not included)

1CR3032 3V battery for wireless Sensor Pad (included)

Caution: To reduce the risk of electric shock, do not expose this product to rain or moisture.

Danger of explosion if batteries are incorrectly replaced. The battery for the Sensor Pad is a user replaceable Lithium (not Li-Ion) battery and is NOT rechargeable. Only use the Angelcare® recommended replacement battery. To obtain information on replacement batteries please contact Angelcare® customer service: www.angelcarebaby.com or contact your local distributor. Instructions will be included with the replacement battery. Keep all batteries away from children.

Batteries:

- DO NOT mix old and new batteries.
- DO NOT mix alkaline standard (carbon-zinc), or rechargeable (Ni-CD, NiMH, etc.) batteries.

• **CAUTION:** RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.

Transmission:

Data transmission between the Nursery Unit and the sensor Pad is made on 2.402 to 2.480 GHz frequency for North America.

Maximum RF Power Transmitted: 4dBm (2.5mW)

RF Channel no.: 79

PROTECT THE ENVIRONMENT

The wheellie bin symbols mean that according to local laws and regulations your product and/or its battery shall be disposed of separately from household waste. When the product/battery reaches its end of life take it to a collection point designated by local authorities. The separate disposal and recycling will help conserve natural resources and protect human health and the environment. Refer to User's Guide for removal of batteries.

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